## CHORD-X™ CHORDAL SIZER INSTRUCTIONS FOR USE

### 1. DESCRIPTION

The Chord-X™ Chordal Sizer (see Figure 1) is designed to assist the surgeon in determining the proper length of chordal replacement prostheses needed to repair the mitral valve. It may be used to help the surgeon select a preconfigured prosthesis, or to easily configure the correct size using straight suture material, during mitral valve repair.

### 2. INDICATIONS

Chord-X™ Chordal Sizer is indicated for sizing or configuring chordal replacement prostheses during mitral valve repair.

### 3. CONTRAINDICATIONS

None known.

### 4. PRECAUTIONS AND WARNINGS


Successful use of this device requires appropriate training and experience in cardiac surgical techniques including mitral valve repair. The replacement of chordae tendinae requires accurate sizing to ensure satisfactory functioning of the valve mechanism post repair or replacement. Chordae tendinae replaced with sutures that are either too long or too short can result in failure to correct valvular incompetence.

*The device has been sterilized using gamma irradiation. It is provided sterile unless the integrity of the package has been compromised. Do not use the device if the package has been compromised. Do not re-sterilize. Do not Reuse. Reuse may result in infection, serious injury, or patient death.*


### 5. RECOMMENDED TECHNIQUES

*Note: These instructions only cover the sizing step of chordae tendinae repair. Please see the Chord-X ePTFE Suture or Chord-X Pre-Measured Loops Instructions for Use for further instruction.*

- Use the blue leaflet hook to support the edge of the leaflet in the planned position.
- Use the thumb-slide to extend the papillary hook to the planned site of attachment to the papillary muscle as shown in Figure 2.
- The leaflet hook may be rotated to any desired angle relative to the papillary hook to improve visibility. Rotate the leaflet hook by using the grips at the base of the barrel.
- The colors of scale match the label colors for the corresponding Chord-X Pre-Measured Loops prostheses.
- The scale number indicates the size of chordal loops produced by tying suture around the two hooks. The scale is accurate to  $\pm 1$ mm. Rotate the barrel so that the leaflet hook is aligned with the papillary hook prior to tying loops. After tying, pull the papillary hook back to remove tied loops from the hooks.
- Once the artificial chordae tendinae are in place, a saline test is recommended to verify that the chordal loop size is correct and that the valve is competent.

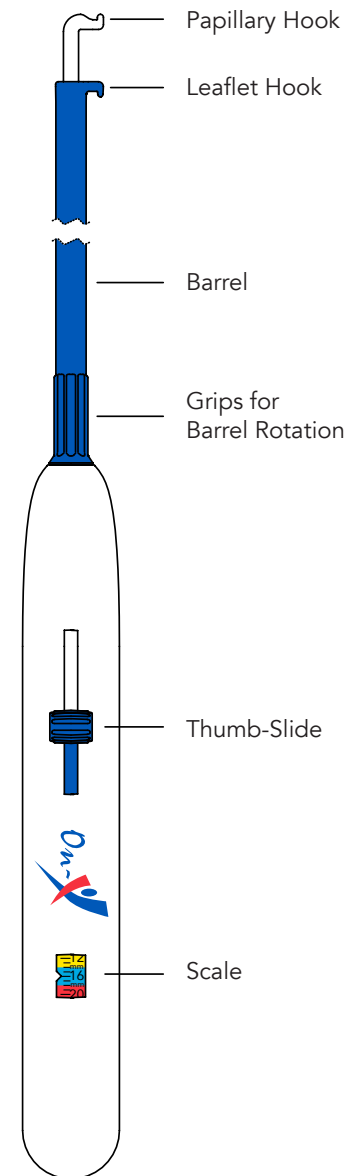**Figure 1:**  
**Chordal Sizer**

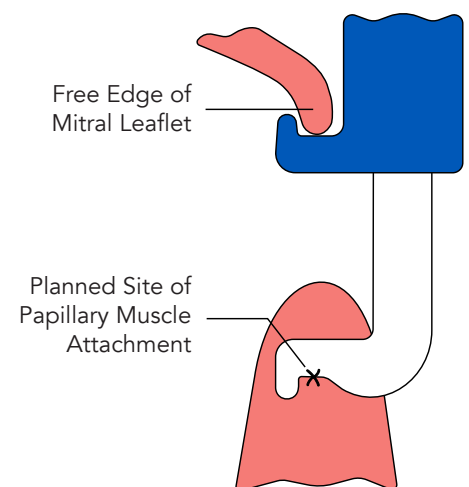**Figure 2:**  
**Sizing**

**STERILE R** Gamma

**REF** CXCS

**On-X Life Technologies, Inc.**

1300 East Anderson Lane, Bldg. B, Austin Texas 78752 U.S.A.

TEL: (512) 339-8000 FAX: (512) 339-3636

WEB: www.onxlti.com EMAIL: onx@onxlti.com

**CryoLife Europa, Ltd.**

Bramley House, The Guildway, Old Portsmouth Road, Guildford  
Surrey GU3 1LR, United Kingdom

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